



ES-10xxSV Line

Bench top steam sterilizers - Class B





e-mail: info@pht.com.bd web: www.pht.com.bd



ES-10xxSV Line



Bench top steam sterilizers - Class B





Sterilization temperature 110°C to 135°C EN-13060-2 class B sterilization cycle

Pre vacuum - water filling - heating - Sterilizing - ventilation- post vacuum drying - end cycle.

Pre vacuum air removal and Post vacuum Drying system

Pre-vacuum for effective air removal that results in increasing sterilization performance. Post-vacuum for drying with remain ing steam removal.

Microprocessor program controller with self auto-tuned and LCD monitor

7 pre-programmed cycle as 2 wrapped, 2 unwrapped, liquid, Bowie- dick and 1 user programmable course.

Full automatic operation system

Easy operation with full automatic process cycle by microprocessor controller from beginning to end cycle.

Self - diagnostic and Safety system

In case of any malfunction, it alarms and displays error codes for easy maintenance, being stop working with automatic ventilation process for safety purpose.

Flushing cycle sterilization system

can be supplied as optional order.

Air filtration cooling system

For the process of drying cycle, 0.2ttm HEPA filtered clean air flows into the chamber in order to protect air contamination and to cool down below 60°C.

Safe and Efficient water supply system

Automatic water supply mechanism which supplies a fixed volume of water to the chamber for maximum efficiency.

Automatic and manual door lock system

When the sliding handle is entered into the locking ring, the door automatically locks after two seconds. The door can be manually opened and locked using the locking wheel system.

Space saving compact design

Maximize working capacity with square chamber and minimize foot space with dedicated engineering design.

Safety Mechanism

- Over pressure release valve
- Over temperature protector with Thermostat
- Higher temp set protector by Digital high alarm limit
- Visible and Audible alarm system
- Automatic dual door locking system





Efficient & Convenient space provided by the movable stand





Microprocessor Control System

LCD Monitor Display

Status - Model and progress status **Time** - Present time and process time

Mode - Set mode

Pressure - Present and process pressure **Temperature** - Present and process temp **Water level** - Present water level

Door - Open, close

Class B Sterilization cycle





Programmable Sterilization Cycles:

MODE	Temperature (°C)	Pressure (kgf/cm²)	Sterilizing time (min)	Drying Time (min)
Unwrapped-1	121	1,1	20	30
Unwrapped-2	134	2,1	5	30
Wrapped-1	121	1,1	30	30
Wrapped-2	134	2,1	15	30
Liquid	121	1,1	20	0
Bowie Dick	134	2,1	4	5 ~ 10
User Program	110 ~ 135	0,4 ~ 2,3	4 ~ 30	0 ~ 60







TECHNICAL SPECIFICATIONS						
Cod.		ES-1011SV	ES-1021SV	ES-1031SV		
Dimensions (WxDxH mm)	Inner	410x560x345	490x655x460	510x690x545		
	Outer	200x300x200	270x405x230	300x455x300		
Capacity		12L	25L	40L		
Heater		2kW	2.5kW	3kW		
Chamber Type		Reinforced and electro polished square chamber				
Material		Stainless Steel 304 standard (STS 361 L option)				
Door lock mechanism		Automatic wheel inter locking with one touch sliding handle				
Vacuum Pump		Built-in diaphragm vacuum pump 25 LPM capacity				
Controller		Microprocessor Program Controller with LCD Monitor				
Sterilization	Temp. Range	110°C - 135°C				
	Pressure Range	0.4-2.3 kgf/cm2				
	Course	Unwrapped 1&2/Wrapped 1&2 / Liquid / Bowie-Dick / User program mode				
Reservoir		6L stainless steel tank with top water filling inlet and silicon closer				
Safety		Over pressure release valve, Over temperature protector Visible and Audible alarm system, Automatic dual door locking system				
Tray		180x260mm	250x360mm	290×400mm		
		Wire tray (Standard) / Perforated tray (Option)				
Electric Supply		110V,60Hz or 220V, 50/60Hz,1Phase				

Optional Accessories



Printer



Perforated Tray



Wire Tray



Pulse Healthcare Technology House: 11/1, Shahid Minar Road Kallyanpur, Dhaka-1216, Bangladesh. Mobile: +8801708008061 e-mail: info@pht.com.bd web: www.pht.com.bd



ESSE3 srl, Via Garibaldi 30 14022 Castelnuovo D.B. (AT) Tel +39 011 99 27 706 Fax +39 011 99 27 506 e-mail esse3@chierinet.it
e= 5 6 3 web: www.esse3-medical.com

